


TiMESH™

The titanized mesh for medical treatment
of inguinal and incisional hernias

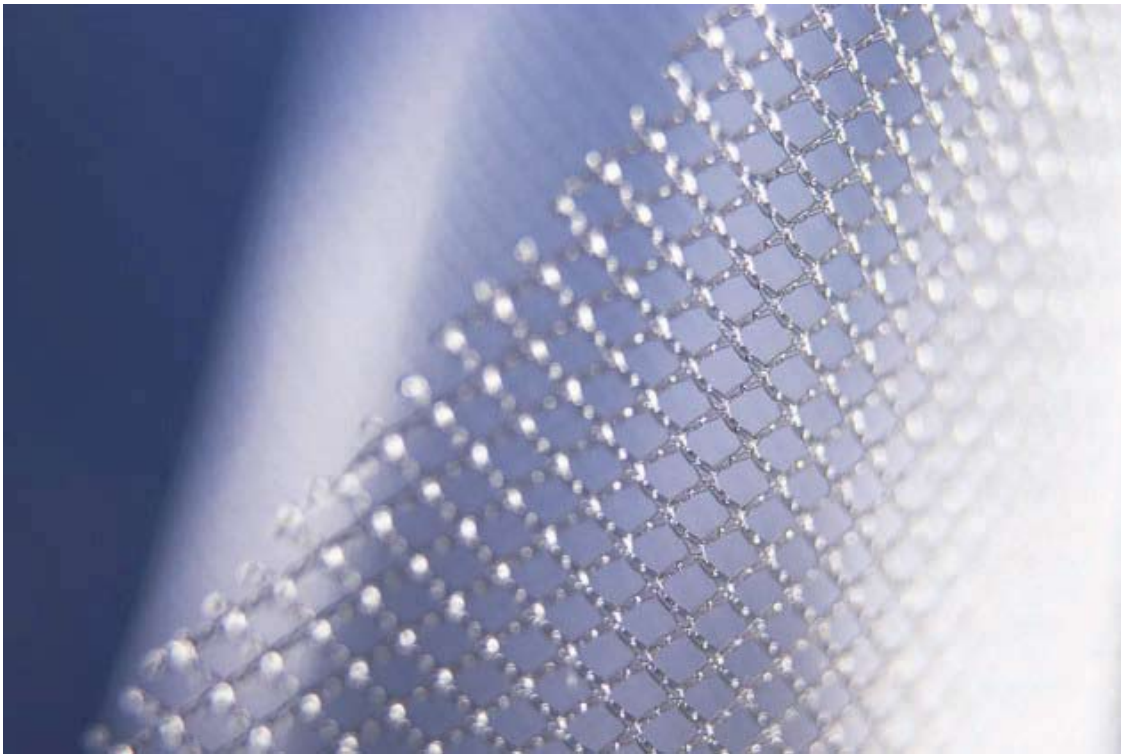

- Biocompatibility
- Ultra-light-weight mesh
- Minimized shrinking¹
- Sufficient flexibility
- Titanized Polymers

TiMESH™ Titanized soft tissue reinforcement implant for hernia repair

Application

TiMESH™ is intended for various indications regarding the use of soft tissue reinforcement implants, e.g. medical treatment of inguinal and incisional hernia.

Design

TiMESH™ is especially designed for:

- all state-of-the-art mesh-surgery techniques
- inguinal, incisional, umbilical, and parastomal hernia repair
- intraperitoneal and peritoneal use

Material

- titanized polypropylene
- pore size ≥ 1 mm
- prosthetic mesh
- tensile strength ≥ 16 N/cm
- monofile fiber
- laser-cut edges

With a grammage of only 16 g/m TiMESH™ meets the high standards of modern, patient-oriented hernia surgery.

Implantation procedure

For inguinal hernia treatment TiMESH™ can be used in both, open and laparoscopic implantation (e.g. TAPP, TEP). For incisional hernia treatment TiMESH™ can be implanted as follows: (a) in the peritoneum, (b) between the abdominal wall and the peritoneum or (c) before the deep muscle layer.

Description	Size	PU	REF
TiMESH™ extralight (16 g/mi)	10 x 15 cm (4" x 6")	3	6000137
TiMESH™ extralight (16 g/mi)	15 x 15 cm (6" x 6")	3	6000311
TiMESH™ extralight (16 g/mi)	20 x 15 cm (8" x 6")	3	6000248
TiMESH™ light (35 g/mi)	10 x 15 cm (4" x 6")	3	6000138
TiMESH™ light (35 g/mi)	15 x 15 cm (6" x 6")	3	6000312
TiMESH™ light (35 g/mi)	20 x 15 cm (8" x 6")	3	6000140
TiMESH™ light (35 g/mi)	30 x 30 cm (12" x 12")	1	6000139
TiMESH™ strong (65 g/mi)	10 x 15 cm (4" x 6")	3	6000470
TiMESH™ strong (65 g/mi)	15 x 15 cm (6" x 6")	3	6000471
TiMESH™ strong (65 g/mi)	20 x 15 cm (8" x 6")	3	6000423
TiMESH™ strong (65 g/mi)	30 x 30 cm (12" x 12")	1	6000424

Reference

1. Schug-Pass *et al.* Surgical Endoscopy. A lightweight polypropylene mesh for laparoscopic repair intraperitoneal repair fo abdominal wall hernias: comparison of biocompatibility with the dualMesh in an experimental study sising the porcine model. *Surgical Endoscopy*. 20: 402-409, 2006.

Exclusive Distributor: Biomet Biologics

Manufactured by:

Certified by: ISO 13485 : 2003

GfE Medizintechnik GmbH

Hoefener Strasse 45 · 90431 Nuremberg, Germany


Phone: +49 (0)911 9315-601 · Fax +49 (0)911 9315-650

info@gfe-medical.com · www.gfe-medical.com

All trademarks herein are the property of Biomet, Inc. or its subsidiaries unless otherwise indicated.

This material is intended for the sole use and benefit of the Biomet Biologics sales force and physicians. It is not to be redistributed, duplicated or disclosed without the express written consent of Biomet.


For product information, including surgical techniques, indications, contraindications, warnings, precautions and potential adverse effects, see the Biomet Biologics website at www.biometbiologics.com

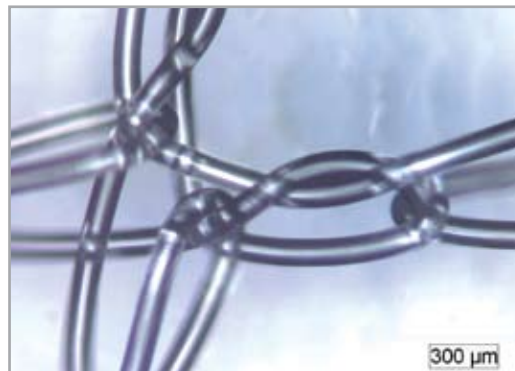

The smooth and flexible fiber (only 16 g/m²) provides optimal biocompatibility and patient comfort, without compromising strength.



Rounded smooth fiber ends, resulting from laser-cut mesh edges, help to prevent micro tissue trama.

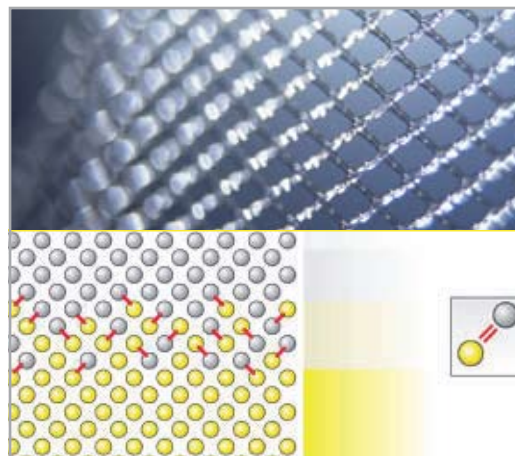

Excellent biocompatibility through a unique combination of a compound material with covalent bonded titanium layer of ~ 30 nm thickness only and a lightweight open porous mesh structure.

BIOMET
BIOLOGICS

One Surgeon. One Patient.™

P.O. Box 587, Warsaw, IN 46581-0587 • 800.348.9500 x 1501
©2009 Biomet Orthopedics • www.biomet.com
Form No. BBI0029.0 • REV093009