

CELL FACTOR
TECHNOLOGIES, INC.
A BIOMET COMPANY

Graft Preparation System



Intelligent Simplicity

Graft Preparation



Remove the plunger from the graft preparation chamber and place funnel into the top.



Fill chamber with desired amount of graft material. Cylindrical portion of chamber is approximately 12ml.



Lightly compact the graft with the plunger. Once finished, lift plunger up until the black rubber tip is above the side port.



Attach syringe with fluid onto the side port using blending connector. Dispense liquids into the graft preparation system and then remove syringe from side port.



Apply light pressure to the plunger for 15 seconds to compress the liquid media into the graft. Let graft stand for two to four minutes before removing from chamber.



Remove end cap and use the plunger to extrude the graft into sterile pan.

Ordering Information

Graft Preparation System	
Description	Catalog Number
CFT™ Graft Preparation System	800-0300

