



MALLORY-HEAD
HIP · PROGRAM[™]
POROUS PRIMARY SYSTEM

BIOMET[®]
ORTHOPEDICS

SINCE 1987

UNCHANGED DESIGN, UNSURPASSED CLINICAL RESULTS, UNCOMPROMISING TECHNOLOGY

A fusion of tapered and finned geometry, the Mallory-Head® Porous Primary System replicates near normal bone stresses and can provide for long-term stability and pain relief. Backed by more than 20 years of excellent clinical performance, the advanced design concepts of the Mallory-Head® System have demonstrated more than 99% survivorship with no osteolysis in numerous clinical studies.^{1,2,4,5}

Three Degree Bi-Planar Taper

Promotes physiologic proximal load transfer and exceptional implant-to-patient fit

Interlok® Finish

Allows for bone ongrowth in the diaphysis

Smooth Finish

Promotes an absence of bone ongrowth, aiding in the gradual off-loading of stresses





Proximal Finned Geometry

Encourages circumferential stress transfer to the femur, while resisting rotational and bending forces

PPS® Porous Plasma Spray

Allows for initial scratch-fit stability and bone ingrowth⁶

UNMATCHED CLINICAL RESULTS


99.2% 99.2% survivorship at a follow-up of up to 18 years in 249 patients 40 years and younger, no osteolysis¹

99.5% 99.5% survivorship at an 11 year follow-up of 188 patients, no osteolysis²

99.2% 99.2% survivorship at a 5–10 year follow-up in 451 patients, no osteolysis⁴

99.0% 99.0% survivorship at a 10–13 year follow-up in 307 patients, no osteolysis⁵

PROVEN OFFSET PHILOSOPHY

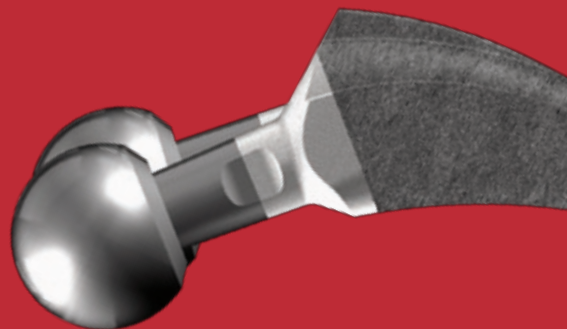

Femurs with greater horizontal offset often have a more varus neck shaft angle. For example, as stem size increases, the offset required also increases.³ Based on this philosophy, the Mallory-Head® offset stem decreases the neck shaft angle, medially shifting the trunion and increasing neck length, while not affecting leg length.

MALLORY-HEAD® LATERALIZED OFFSETS

Stem Size	Additional Offsets
8–10mm	6mm
11–14mm	7mm
15–17mm	8mm

REFERENCES

1. Mallory-Head Primary Press-Fit Replacement. Presented at AAOS, 1996.
2. Bourne, R.B. *et al.* Tapered Titanium Cementless Total Hip Replacements a 10- to 13-Year Follow-Up. *Clinical Orthopaedics and Related Research*. 393: 112–120, 2001.
3. Head, W.C. Mallory, T.H. Emerson, Jr., R.H. The Proximal Porous Coating Alternative for Primary Total Hip Arthroplasty. *Orthopedics*. 22: 813, 1999.
4. Hoffman, A.A. Response of Human Cancellous Bone to Identically Structured Commercially Pure Titanium and Cobalt Chromium Alloy Porous-Coated Cylinders. *Clinical Materials*. 14: 101–115, 1993.
5. Davey, J.R. Tozakoglou, E. The Role of Lateral Offset Stems. *Orthopaedic Transactions*. 22(1): 273, 1999.
6. Emerson, R. *et al.* Effect of Circumferential Plasma-Spray Porous Coating on the Rate of Femoral Osteolysis after Total Hip Arthroplasty. *Journal of Bone and Joint Surgery*. 81: 1291–8, 1999.

For general risk information on hip products, please see Biomet's website.

This material is intended for the Biomet sales force and surgeons only and should not be redistributed without express consent from Biomet.

Mallory-Head® PPS® and Interlok® are trademarks of Biomet Manufacturing Corp.

BIOMET[®]
ORTHOPEDICS, INC.
Driven By Engineering

P.O. Box 587, Warsaw, IN 46581-0587 • 800.348.9500 ext. 1501
©2007 Biomet Orthopedics, Inc. All Rights Reserved • www.biomet.com
Form No. Y-BMT-984/043007/M